



Smart Vets use Smart Products™



Instructions for the
MDC EeziSet™ Cat Trap

A178-315 [regular]
A178-316 [extended]

Introduction

The EeziSet Cat Trap is a self-activating trap, which closes automatically when a cat enters and stands on the treadle plate. The treadle plate is linked to a spring loaded release pin allowing the trap-door and locking bar to drop into place to secure the cat for temporary confinement. MDC highly recommend that traps are monitored and checked at least every 2 hours.

Operation - Fig. 1


Raise the transparent door A and place a quantity of suitable bait adjacent to the treadle plate B. Lower the transparent door A back into place.

With the aid of the external lever C, raise the locking bar until it touches the roof of the trap. Hold the locking bar in position and raise the trap-door D smartly so that it latches onto the release pin E.

The trap is now set and can be placed into position.



FIG. 1

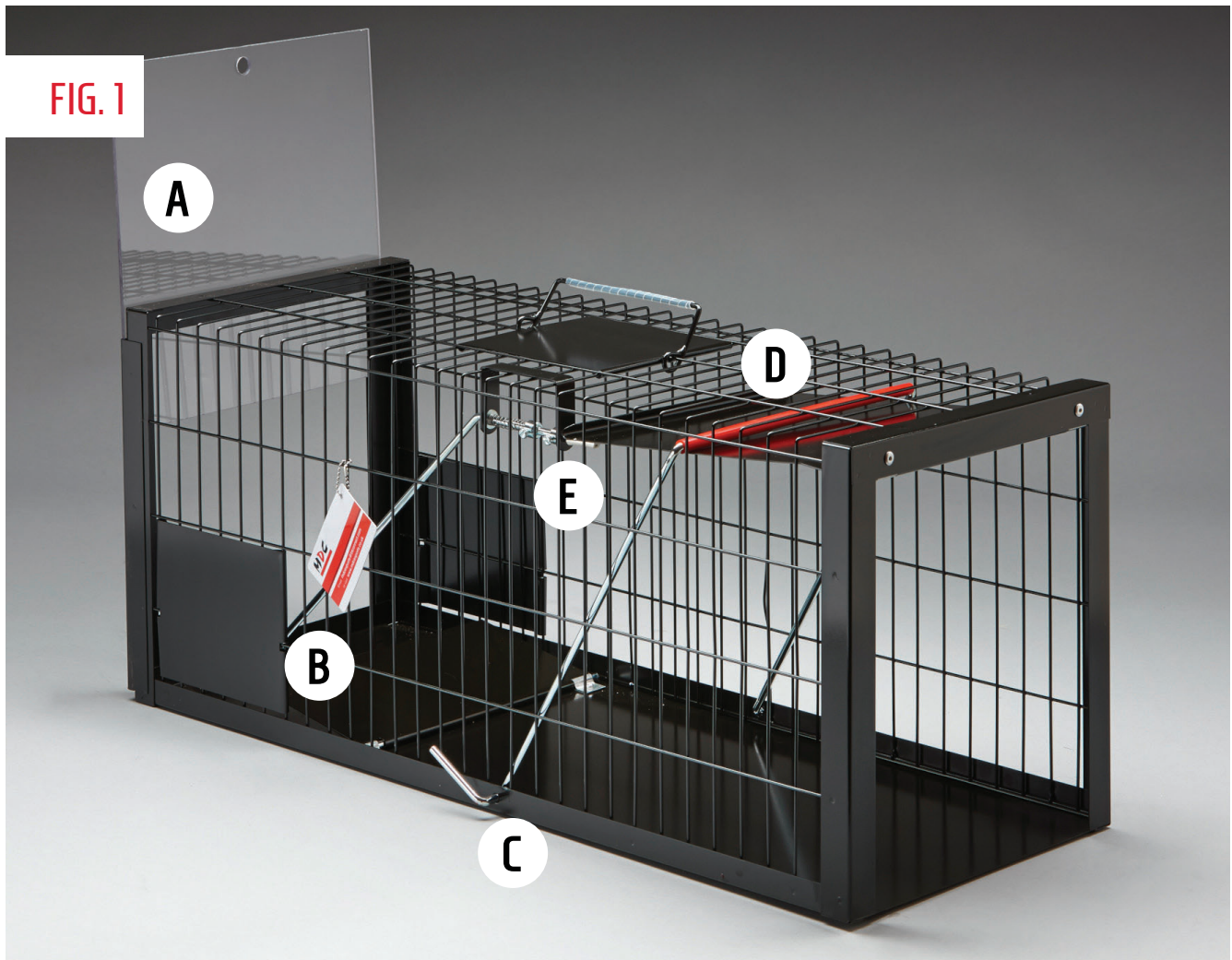

Transferring the cat

Position the trap-door end of the EeziSet Cat Trap against a solid surface. An MDC Trap Transfer basket (A236-354H) is butted up adjacent to the transparent end door. Both doors are raised allowing the cat to be transferred without physical handling. Covering the Trap Transfer basket (MDC cover A236-354D) will also encourage a faster transfer, as the cat is more likely to seek out the darker area for security.

Trapping tips

Practice using the trap (and transfer technique) until you are familiar with it.

Traps should be located where the cats are normally fed. Work with local cat feeders and recruit them as allies/helpers.

Bait the unset trap and allow the cat/s to eat from the trap for several days.

Set traps should be monitored and checked every two hours. Such checks avoid having the cat being held captive too long and also provides an opportunity to ensure that the trap has not been accidentally sprung or interfered with.

Be aware of the cats welfare and minimise stress wherever possible.



Smart Vets use Smart Products™

Get in Touch!



Unit 11, Titan Court, Laporte Way, Luton, Bedfordshire, LU4 8EF



+ 44 (0)1582 655 600



info@mdcexports.co.uk



mdcexports.co.uk



facebook.com/mdcexports

Maintenance of the trap

Keep the trap clean. Periodically lubricate all hinge points and release pin with oil or light grease.

Disclaimer

All advice and recommendations within this instruction leaflet are for information purposes only and are not intended to replace appropriate care from the operator. The operator remains solely responsible for the consequences of their trapping activities and MDC cannot be held liable for any injury to either the operator or the trapped animal. MDC traps are designed for the species stated however other animals may be inadvertently caught. MDC cannot be held liable for any injury to an animal inadvertently caught or any damage to the trap that may occur.

"EeziSet" and "Smart Vets Use Smart Products" are trademarks of MDC Exports Limited.