



Smart Vets use Smart Products™




Instructions for the

MDC Quick Release Dog Grasper™

A106-307-24 / A108-307-36 / A110-307-48 / A109-307-60

Introduction

The The Quick Release Dog Grasper is a slip loop/noose cable attached to a sturdy pole for animal handling professionals to safely retract, restrain and control dogs by gently tightening the loop over an animals head and neck. Using a grasper is an effective way to handle fractious dogs, to rescue a trapped dog or to deal safely and humanely with a dog which is hostile. In these and other situations, the dog can be seized and restrained from a safe distance. The grasper and replacement cables are available in four lengths.

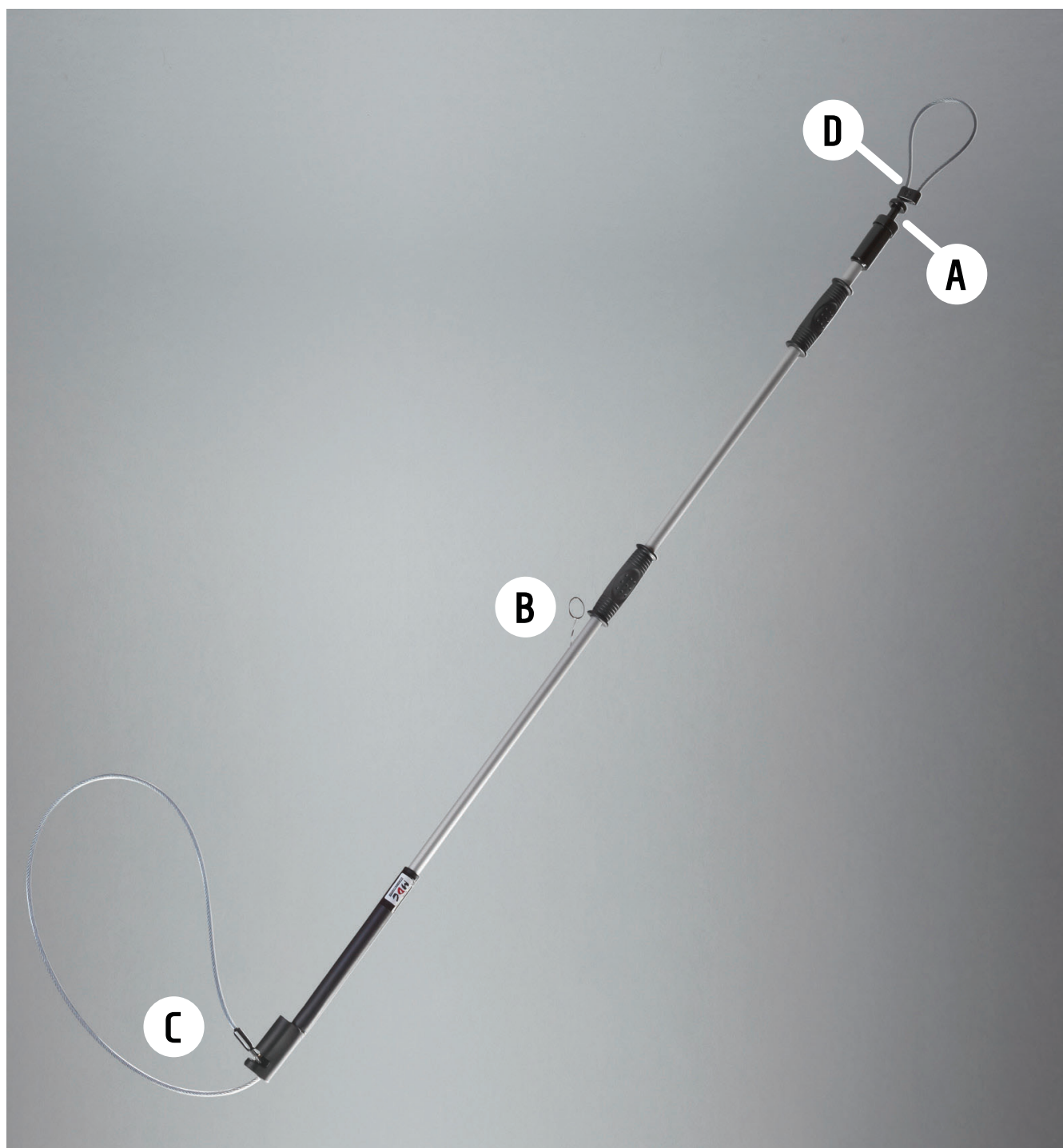

Operation

Pull release knob **A** with one hand and with the other, feed through all of the cable. Pull ring **B** and insert the end cable coil into the retracting pin **C**. Releasing ring **B** fastens the coil in place and the loop/noose is formed. Having placed the loop gently around the dogs' neck, slowly tighten the loop by pulling the cable end. This should only be tight enough to prevent the dog from escaping the loop. The dog may then be held securely until transferred or treated. To release the loop, pull ring **B**.

The Quick Release Grasper is an automatic locking pole. Always practise gently tightening and releasing the loop until you are comfortable with the operation before using on any animal. Only use in confined spaces. Do not attempt to catch dogs on the run. For tips on humane dog handling – see our video on YouTube: MDC Exports Animal Welfare Products

Replacement cables:

Remove **D** metal end clamp from the original cable, pull release knob **A** and extract the cable. Using the same technique, thread through the replacement cable and reattach the end clamp reforming the handle.

Product range

A106-307-24	61 cm Grasper
A111-309-24	Replacement cable
A108-307-36	91cm Grasper
A111-309-36	Replacement cable
A110-307-48	122cm Grasper
A111-309-48	Replacement cable
A109-307-60	152cm Grasper
A111-309-60	Replacement cable



Smart Vets use Smart Products™

Get in Touch!



Unit 11, Titan Court, Laporte Way, Luton, Bedfordshire, LU4 8EF



+ 44 (0)1582 655 600



info@mdcexports.co.uk



mdcexports.co.uk



facebook.com/mdcexports

Maintenance

Keep the grasper clean. Periodically lubricate the release ring with oil or light grease.

Disclaimer

All advice and recommendations within this instruction leaflet are for information purposes only and are not intended to replace appropriate care from the operator. The operator remains solely responsible for the consequences of their dog handling activities and MDC cannot be held liable for any injury to either the operator or the dog.

"Quick Release Dog Grasper" and "Smart Vets Use Smart Products" are trademarks of MDC Exports Limited. Patent Number: 2406032 .